

Cleaning your AeroChamber Flow-Vu



Small mask
0-18 months



Medium mask
1-5 years



Youth mouthpiece
5 years+



Mouthpiece



Adult Small mask




Adult mask


CHILD

ADULT


Instructions:

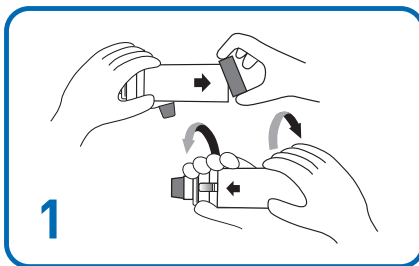


1. Take your AeroChamber apart before cleaning.

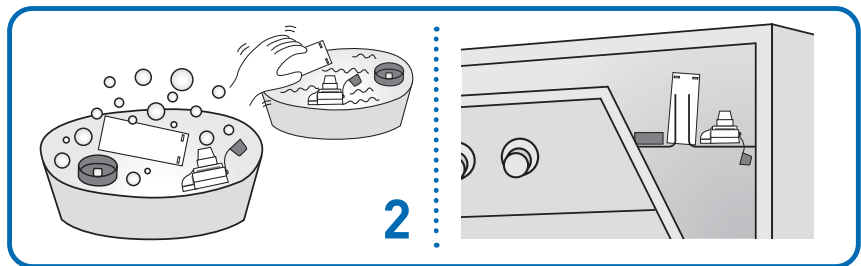

2. The parts can be soaked in luke-warm soapy water for 15 minutes. Rinse thoroughly afterwards. The AeroChamber is also dishwasher safe up to 70°C.

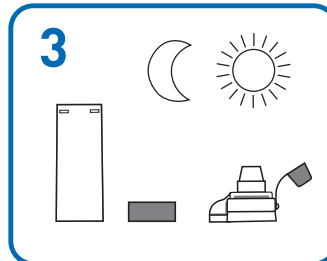

3. Allow the parts to air dry or dry in the dishwasher. The parts should be completely dry before you use them.

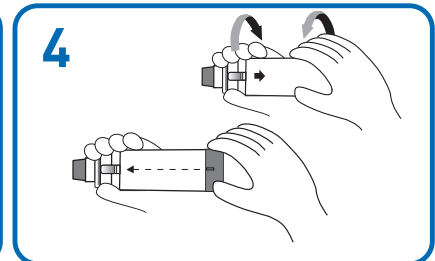

4. Reassemble the chamber as shown. Centre the alignment feature on the backpiece with the Flow-Vu indicator.